

## Sterile Tamper-Evident Luer Lock Tip Cap

Three-part plastic cap used to indicate tampering on a luer lock syringe.

### Specifications:

- Cap: polypropylene, natural
- Sleeve: medium-impact polystyrene, red
- Package: polypropylene, natural
- Made in the USA



### Contraindications:

None when used as described in product insert.

### Instructions for Use

1. Peel back tip cap tray lidding.
2. Place a luer lock syringe over a tip cap in tray. Push syringe firmly onto tip cap and twist (Figure 1).

**Note:** Tip cap must be firmly attached to effectively indicate tampering.

3. To remove cap, grip outer sleeve of tip cap and pull away from syringe (Figure 2).
4. Twist off tip cap counter-clockwise and remove inner sleeve (Figure 3).

This product may be covered by one or more of the following U.S. patents or by pending patents: U.S. No. 6,585,691; U.S. No. 6,921,383; U.S. No. 7,762,988

	Do not re-use		Non-pyrogenic
	Do not re-sterilize		Consult instructions for use
			This device is not made with natural rubber latex
	Do not use if package is damaged		Sterilized using ethylene oxide