

Surgical Masks, Package of 10



Item #: PS345
UPC: 063636 003456

Please note: Masks are blue

- Elastic Ear Loops
- Disposable
- Pleated design for maximum comfort
- Can be worn with glasses
- Colour: blue
- Lightweight
- Fluid resistant
- Easy to use
- 10 per package
- Easy to breathe through
- Protection from non-toxic dust and contaminants
- Ideal for blood-borne pathogen hazard spill kits
- For use in medical clinics and by dental hygienists

Instructions for proper use:

1. Before wearing mask, wash your hands thoroughly.
2. Place mask against face so that the white side is in contact with your face and the side with metal/plastic strip is upwards
3. Loop the bands over and behind the ears for holding mask firmly in place.
4. Pinch the flexible metal/plastic strip to mould the mask to the bridge of the nose.
5. Open pleats of mask to extend to the chin

****Final Sale****

